


Programmable

Microplate/Microstrip Washer

- Automatic alignment
- Automatic volume calibration
- Automatic or manual depth control
- Many wash options

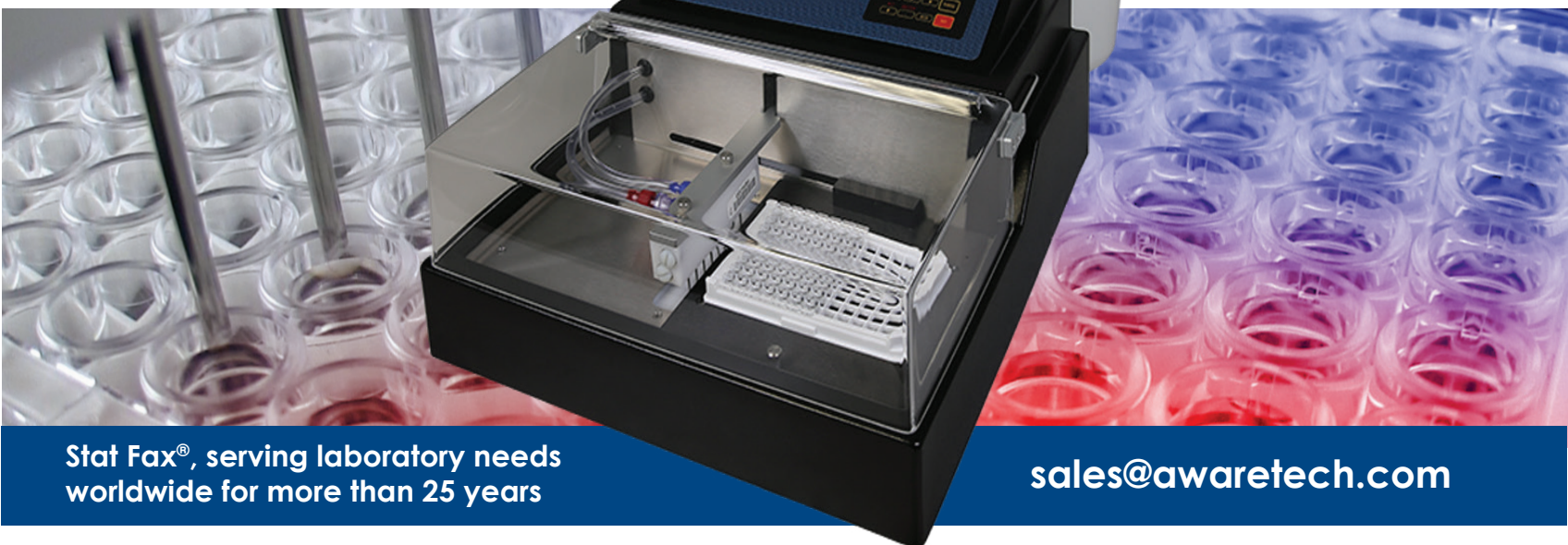

**Stat Fax®, serving laboratory needs
worldwide for more than 25 years**

sales@awaretech.com

The Stat Fax® 2600 standalone programmable microwell plate and microwell strip washer is suitable for use with standard 96-well microwell plates. The easy to use touch-pad and LCD screen make programming and executing wash and rinse protocols straight forward in this reliable and versatile microwell plate washing system.

Stat Fax® 2600 can mix, soak, and maintain constant timing despite run size. Aspiration position is optimized for each well type. Double aspiration also ensures complete emptying. The software automatically adjusts for different plastic-ware.

Features of the Stat Fax® 2600

- Washes flat or round bottom microwells
- Automatically positions each row for washing
- Stores 50 user programmable wash protocols
- 6 factory programmed wash/rinse modes
- Aerosol shield & aerosol line filter (biohazard protection)
- Strip selection for partial plate washing
- Low-maintenance
- Economical open system/compact design
- (Optional extra wash bottle)

STAT FAX[®] 2600

Microplate Washer

Residual Volume	<3 µl
Dispense accuracy	<3 % CV at 300 µl
Processing time (full plate)	55 seconds aspirate/dispense with single aspirate
Liquid contact materials	Halogen lamp with lamp saver feature
Display	Two-line LCD, alphanumeric, 24 characters per line
Keypad	16-key, domed membrane switch 4 x 4, enunciating
Non-volatile memory	Battery supported RAM, holds approx. 50 wash protocols
Washing programs	Aspirate single or double , dispense variable speed and volume, mix, soak - up to 99 minutes 99 seconds
Special menu options	Auto alignment - senses physical geometry of the wash head, plate, and carrier mechanism Standby - disables pumps, releases vacuum and pressure Self Check - performs internal performance test and auto alignment Auxiliary - selection of well type, select auto/manual well depth and dispense depth, constant plate cycle time, viewing of preprogrammed tests, configure auto-rinse, set the time and date Dispense volume auto calibration
Dimensions & weight	15.5" x 13.5" x 7.5" (39 x 34 x 19 cm) Weight 19 lbs (8.6 kgs)
Bottles	Wash & waste 2 L, rinse 1 L, plastic with volume sensor probes
Certifications	NRTL listed, CE mark, ISO 13485:2016
Optional 2600 Plus model	Additional 2 L wash bottle and additional solenoid installed, for added versatility and convenience. (Must be requested at the time of the order)



Awareness Technology, Inc. USA & INTERNATIONAL:
1935 S.W. Martin Hwy., Palm City, FL 34990, USA
tel.: +1 772 283 6540, fax: +1 772 283 8020
info@awaretech.com

Awareness Technology, Inc. EUROPE:
Franz-Siegel-Gasse 1, 2380, Perchtoldsdorf, Austria
tel.: +43 (1) 804 81 84, fax: +43 (1) 804 81 85
info@awaretech.eu

Awareness Technology, Inc. ASIA & AFRICA
Q3-146, SAIF Zone, 120543, Sharjah, United Arab Emirates
tel.: +971 (06) 557 80 58, fax: +971 (06) 557 80 59
info@awaretech.com



Pictured with optional wash bottle

ISO 13485:2016

